

What is it?

The MedReady™ medication reminder and dispensing system is designed to enhance medication adherence. The pill dispenser offers users a sense of comfort knowing that their medications are conveniently organized and that they will be gently prompted to take them at the appropriate times.

How does it work?

The MedReady™ is set up with dispensing times and notification preferences. Then, a family member, caregiver, or pharmacist fills the dispensing cups with medication.

When it's time to take a medication, the appropriate compartment alarms the user with audible and visual reminders.

If the medication is not removed within the assigned time frame, designated recipients will receive automatic notifications via phone, text, and/or email.

Who is it for?

MedReady™ is ideal for an array of people who: have multiple prescriptions, difficulty organizing pills, a hard time remembering, dexterity issues, difficulty reading labels, risk of mis-dosing and for anyone who needs the convenience of updating regimens and configuring the type and delivery method of notifications.



Features

- Secure tamper resistant lock and key to ensure the utmost protection.
- Loud reminder prompts blink or buzz to remind the user at designated dispensing times.
- The 28 compartments easily accommodates 9 standard pills allowing flexibility for dispensing up to 4 doses a day for 7 days, 3 doses a day for 9 days, 2 doses a day for 14 days, or 1 dose a day for 28 days.
- Refill trays can be filled ahead of time by family members, a pharmacist, or caregiver, and placed directly into the MedReady™ medication dispenser.
- Dispensing compartments only open at designated times to prevent mis-dosing or overdosing.
- Users or authorized caregivers can remotely configure or update dispensing times, reminder prompts, and notifications via the convenient MedReady™ Portal.
- Notifications can be received via phone, email, and text notifications.
- Users or caregivers can remotely access detailed device activity reports including compliance information.
- In the event of a black out or short term travel, the back up battery holds power for 48 hours.

